

aqua BOOST

PRO

by Prosana™



AquaBoost Pro Training Manual

universalcompanies.com/AquaBoostPRO

What is AquaBoost?

- **Non-invasive hydrodermabrasion** facial equipment that provides exceptional exfoliation, extractions, and hydration.
- Remarkably **smooth, radiant skin** in just 3 steps!
- **Safe** form of epidermis exfoliation, sebum reduction, blemish extractions, and product penetration
- **Custom treatments** with easily adjusted levels of solution flow and suction



CONTENTS

What Makes AquaBoost Unique page 03

Benefits of AquaBoost. page 04

AquaBoost Contraindications page 05

Client Frequently Asked Questions page 06

Serum 1 Cleanse & Exfoliate page 07

Serum 2 Deep Cleanse. page 08

Serum 3 Soothe & Hydrate page 09

AquaBoost Serums & Modalities page 10

Using Your AquaBoost page 11

Using Your AquaBoost - Treatment Wand page 12

Using Your AquaBoost - CryoWand page 13

Caring for your AquaBoost page 14

Print Ready Cleaning Log page 15

Treatment Protocols page 16

Safety Data She page 31

What Makes AquaBoost Unique?

Patented Treatment Tip

The patented treatment tip is non-abrasive and has a unique dome shape that glides along the skin. It performs the physical exfoliation your guests know and love but without the irritation. AquaBoost's treatment tip is safe for all skin types and skin conditions, even the most sensitive skin. What's more, you're not required to change treatment tips during an AquaBoost service.



Cryotherapy with Blue LEDs

Not found on other hydrodermabrasion systems, the CryoWand is unique to the AquaBoost machine. This round, handheld wand completes your AquaBoost service by providing multiple benefits, is safe for other areas of the body, and can be incorporated into other services.



Cryotherapy

- Minimizes any redness by encouraging vascular movement back through the skin.
- Helps to close the pores and promote firm, glowing skin

Blue LED Lights

- Helps to destroy bacteria remaining on the skin from extractions
- Cools and comforts the skin post AquaBoost service

Serums

You know your guests' skin better than anyone. So, we created a safe, highly efficacious serum system that allows you to select which serums are best. Containing beneficial natural source ingredients, these serums provide hydration, exfoliation, and antioxidants to nourish skin while keeping its integrity top of mind. You'll select Serum 1, Serum 2, or both to help you meet your guests' skin care goals.



No Downtime

Pairing our patented treatment tip with our custom serums, AquaBoost provides a safe, effective service without downtime. The gentle nature of the treatment tip allows for more frequent services. And our carefully selected ingredients produce results without stripping the skin, meaning there's less chance of irritation or reaction.

AquaBoost services can be received without concern of skin shedding or peeling, and without the level of redness seen in other types of dermabrasion services. Guests can return to their day without disruption.



Benefits of AquaBoost

AquaBoost can be performed as a stand-alone treatment or as an upgrade. However you choose to incorporate AquaBoost, guests will experience multiple benefits.

Reduces the appearance of fine lines & wrinkles

Skin shows signs of aging as the body's natural processes begin to slow. Skin also appears to be "aging" when we lose a healthy glow. By removing dead skin cells on the epidermis and infusing skin with hydrating ingredients, it looks vibrant, healthy, and youthful.

Exfoliates for smooth, radiant skin

AquaBoost delivers superior physical and chemical exfoliation. Its serums are introduced to the skin to gently loosen the cellular bond of dead skin cells. The treatment tip then provides gentle, effective friction to physically remove dead skin cells, resulting in luminosity.

Removes impurities

AquaBoost combines custom formulated treatment serums with user adjustable vacuum pressure to gently extract impurities from the skin.

No downtime or discomfort

Natural source ingredients and gentle humectants provide effective treatments with less risk of skin sensitivity. Exfoliation is achieved with a combination of physical and chemical forms and doesn't cause the skin to shed for days or weeks following a treatment. Instead, dead skin cells are gently removed and swept off the skin immediately, leaving only healthy, glowing skin tissue.

Deeply hydrates skin

AquaBoost infuses ingredients like hyaluronic acid, aloe, lactic acid, and ceramide NP. Serum is delivered to the skin with a pressurized, user adjustable treatment wand. This allows the products to penetrate the skin at a deeper level than pressing with hands alone can provide.

Minimizes post-treatment redness with cryotherapy

The CryoWand provides a cooling touch to the skin which comforts the skin post treatment and promotes healthy blood flow being pushed back through the skin. This can result in less redness post treatment.





AquaBoost Contraindications

While AquaBoost is a safe and effective treatment, if you question a guest's ability to have an AquaBoost facial, it is recommended you follow your company policy to determine if performing a treatment is appropriate.

Pregnant or Nursing

It is recommended that guests check with their doctor before receiving any type of treatment.

Medication

Guests currently taking Accutane, or other topical/oral medications that can cause fragile and/or thinning skin, should not receive the AquaBoost treatment until cleared by their doctor.

Open Skin Lesions

Areas of the skin that have open lesions should be avoided as to not cause tearing or additional skin trauma (e.g., acne, rosacea, etc.).

Sensitive Skin

Active rosacea may not be suitable for AquaBoost treatments.

Sensitized skin that reddens easily can receive treatment. It's important to adjust your settings as needed for your guest comfort and results.

NOTES



Client Frequently Asked Questions

Clients may ask you questions about the AquaBoost treatment, so we created some FAQs for you.

Question:	Response:
What is AquaBoost?	<i>A hydro dermabrasion system that effectively cleanses, exfoliates, and infuses hydration for glowing skin!</i>
Can I have an AquaBoost service if I have sensitive skin?	<i>Yes! AquaBoost is safe for all skin types.</i>
How Does AquaBoost help with acne?	<i>AquaBoost removes the dead skin cells that trap dirt and oil while its cleansing serums dissolve impurities and promote radiance. Your skin can then return to its natural function, softer and looking healthier and clearer.</i>
My skin gets red easily! Will I be able to return to work after my service?	<i>Absolutely! AquaBoost is effective yet gentle so there's no downtime. And every AquaBoost service concludes with our unique CryoWand which cools and calms the skin. This encourages continued vascular activity, resulting in less redness.</i>
How long is the service?	<i>AquaBoost takes just 15 minutes! It's a perfect service upgrade OR stand-alone service. It's quick and can even happen on your lunch break.</i>
If I have active rosacea, can I receive an AquaBoost Service?	<i>It is recommended that you wait until the flare-up subsides.</i>
I just received a peel 2 weeks ago. Can I receive an AquaBoost service?	<i>It is recommended that you wait at least 4 weeks after a peel before receiving an AquaBoost service.</i>
Are there any instructions I need to follow after my AquaBoost service?	<ul style="list-style-type: none"> • Refrain from any retinol or home use acids, such as salicylic or glycolic, for the remainder of the day of service. • Apply sunscreen daily. • Enjoy your glowing skin!



Serum 1

Cleanse & Exfoliate

Ingredients:

Aqua, Aloe Barbadensis Leaf Extract, Glycerin, Glycolic Acid, 1,2-Hexanediol, Caprylyl Glycol, Rosa Damascena Flower Water, Lactic Acid, Sodium Hydroxide, Salicylic Acid, Sodium Hyaluronate, Camellia Sinensis (Green Tea) Leaf Extract, Sodium Benzoate

Features:

5% Aloe Leaf Extract:

Soothing, Hydrating

1% Glycolic Acid: Cellular Turnover, AHA

.025% Lactic Acid: Exfoliating AHA, Brightening

.1% Salicylic Acid: Purifying, BHA

.25% Sodium Hyaluronate: Hydrating

Best Suited For:

- Dry
- Normal
- Normal to Oily
- Combination

Benefits:

- Cleanses the skin and gently exfoliates
- Removes excess oils without stripping the skin of beneficial oils
- Removes dry/dead skin cells allowing beneficial ingredients to penetrate the skin
- Hydrates the skin while exfoliating

Usage:

Serum 1 can safely be used for all skin types and will be your most common selection.



PSN130

Approximately 4-8
Treatments / Bottle
(2-4 oz. per treatment)

NOTES

Serum 2 Deep Cleanse

Ingredients:

Aqua, Aloe Barbadensis Leaf Extract, Glycerin, 1,2-Hexanediol, Caprylyl Glycol, Sodium Hyaluronate, Camellia Sinensis (Green Tea) Leaf Extract, Allantoin, Sodium Lactate, Salicylic Acid, Panthenol, Sodium Benzoate, Sodium Hydroxide, Arginine

Features:

20% Aloe Leaf Extract:

Soothing, Hydrating

.03% (3% high molecular weight)

Sodium Hyaluronate: Hydrating

.1% Salicylic Acid: Purifying, BHA

.1% Panthenol:

Hydrating, Decreases TEWL

.25% Allantoin: Soothing

Best Suited For:

- Combination
- Oily
- Acneic / Acne Prone

Benefits:

- Helps to fight bacteria and purify the skin
- Aids in reduction of excess sebum without drying the skin or creating an imbalance
- Calms and soothes the skin while exfoliating and removing impurities
- Aids in exfoliation

Usage:

Serum 2 is your serum selection for guests with excessive sebum or congestion, or acne prone and acneic skin.



PSN131

Approximately 4-8
Treatments / Bottle
(2-4 oz. per treatment)

NOTES

Serum 3

Soothe & Hydrate

Ingredients:

Aqua, Aloe Barbadosensis Leaf Extract, 1,2-Hexanediol, Caprylyl Glycol, Rosa Damascena Flower Water, Sodium Hyaluronate, Chamomilla Recutita (Matricaria) Flower Extract, Laminaria Digitata Extract, Camellia Sinensis (Green Tea) Leaf Extract, Ginkgo Biloba Leaf Extract, Cucumis Sativus (Cucumber) Fruit Extract, Dipotassium Glycyrrhizate, Allantoin, Sodium Benzoate, Lactic Acid, Caprylyl/Capryl Glucoside, Ceramide NP, Arginine, Tocopheryl Acetate

Features:

25% Aloe Leaf Extract:

Soothing, Hydrating

.5% Rose Water: Soothing, Hydrating

.05% Dipotassium Glycyrrhizate:

Brightening, Soothing

.05% Allantoin: Soothing

0.015% (1.5% high molecular weight)

Sodium Hyaluronate: Hydrating

.005% Ceramide NP: Hydrating,
Decreases TEWL

Best Suited For:

All skin types

Benefits:

- Filled with hydrating and moisture-binding properties.
- Calming to help reduce redness following hydrodermabrasion
- Antioxidant properties

Usage:

Serum 3 is used to complete your AquaBoost protocol by infusing hydration and nourishing properties. Serum 3 is included in most AquaBoost protocols and will be used frequently.



PSN132

Approximately 4-8
Treatments / Bottle
(2-4 oz. per treatment)

NOTES

AquaBoost Serums & Modalities

AquaBoost serums are designed for trained professionals to customize each service based on skin analysis. Use 1, 2, or all 3 serums to tailor the service to your guests skin care needs and goals.

Serum 1 Cleanse & Exfoliate	Serum 2 Deep Cleanse	Serum 3 Calm and Soothe
<p>In all AquaBoost protocols, you will find a specific serum choice listed, or the option to choose based on your guests skin.</p> <p>On each product information sheet, you will find a list of who each serum is best suited for to help guide you.</p>		<p>In all AquaBoost protocols, you will find that Serum 3 is used in nearly all treatments as the source of intense hydration. Please note however, some protocols do not include Serum 3 and instead utilize a moisturizer of your choice.</p>

Looking for additional ways to incorporate AquaBoost into all your treatments? AquaBoost allows you to customize your service experience by layering in one, or all, of the AquaBoost Elements. The chart below shows some examples of common skin conditions and concerns and what AquaBoost elements help to address those concerns.

Skin Condition or Concern	Serum 1	Serum 2	Serum 3	CryoWand
Acneic Skin	X	X	X	X
Sensitive	X		X	X
Dry/Dehydrated Skin	X		X	X
Lines and Wrinkles	X	X	X	X
Hyperpigmentation	X	X	X	X
Redness or Irritated Skin			X	X
Sagging Skin	X		X	X
Post Peel				X
Post Extraction			X	X
Excessive Sebum	X	X	X	X
Keratinized Skin	X	X	X	X

These are general recommendations. AquaBoost treatments are intended to be customizable based on client's specific needs.

NOTES

Using Your AquaBoost

Serum Flow & Suction

- Begin your treatment with settings turned completely down to 0.
- Ensure that serum is flowing from the treatment wand prior to gliding on the skin. Do not, at any point, test your treatment wand on your skin. Using a gloved hand, turn the suction down low, and press the treatment wand on your glove to ensure serum is flowing.
- Adjust your suction and serum flow throughout the service based on guest need. Be sure to check in with your guest to gauge their comfort level.
- If you notice your guest’s skin is very “thirsty,” increase your serum flow.
- If you notice the treatment wand “skipping” or releasing rapidly and unexpectedly, turn your suction down and be sure you’re holding the skin taut.
- If your guest is experiencing dripping of serum, turn your serum flow down and your pressure up.
- Keeping your guests skin type in mind, having pressure at 2.5 and flow at 3:00 position is a good starting point.



UCo[®]™

How to Perform
an AquaBoost
Treatment



Get free staff training
on the UCo Learning Network!



NOTES _____

Using Your AquaBoost

Treatment Wand Usage



Begin at the chin
moving upward & outward.

Always hold the skin taut

with one hand and glide your treatment wand with the other hand. This will help to prevent skipping or rapid release of the treatment wand.

Work in sections

and adjust your settings for more stubborn extractions.

When working on the nose:

Always support the opposite side of the nose when applying pressure.

Remember to pop the crease of the nasal labial fold to thoroughly cleanse and extract.



Always keep the top of the treatment tip facing toward the inside of the face.

This will help ensure no skipping or pulling on the skin.

NOTES

Using Your AquaBoost

CryoWand + Blue LED Lights

- Turn on the CryoWand and allow about a minute for the wand to cool.
- Apply a thick gel or serum to the skin. Using a cream mask is **not recommended**.
- Begin at the chin.
- Apply to the skin and move in upward, circular motions.
- Move the wand in the same pattern as your treatment wand, from the inside to the outside of the face.
- You can spend extra time on areas that required more intense exfoliation or extractions to provide additional comfort for the skin.
- The CryoWand can be used on the full face or on a specific area (extraction site, for example). However you choose to incorporate the CryoWand, it is important that you remember to:
 - Keep the wand moving in circular motions and do not allow the wand to sit in one place on the skin.
 - Use a gel or serum to provide slip on the skin.



Note: Depending on your product selection, you may notice some product dry out and find the CryoWand does not glide easily and therefore causes tugging on the skin. In this case, additional product or slip may be required. Applying product to the full face immediately following Serum 3 is highly recommended. At that moment, skin is prepped and ready to receive additional nourishment. Therefore, applying product all at once is recommended.

NOTES

Treatment Preparation AquaBoost Signature Facial

PRODUCTS REQUIRED FOR SERVICE:

SKU	PRODUCT NAME
PSN130	Serum 1: Cleanse & Exfoliate
PSN131	Serum 2: Deep Cleanse
PSN132	Serum 3: Soothe & Hydrate
AQB206	Treatment Tip
NB525	Gloves
PSN126	Prosana Refining Cleanser
PSN127 OR PSN123	Prosana Refining Mist Prosana Hydration Mist
MASK OF YOUR CHOICE	Suggestions: <ul style="list-style-type: none"> • Hyaluronic Acid Sheet Mask (HM4606) • 4-in-1 Full Face Crystal Collagen Gel Mask (HM4551) • Vitamin C Crystal Collagen Gel Mask (HM4569)
PSN116	Prosana Nail & Cuticle Oil
PSN129	Prosana Calming Balm

CONTRAINDICATIONS
<ul style="list-style-type: none"> • Open lesions on the skin. • Active Rosacea flare-up. • Pregnant & nursing women should consult with a physician. • Accutane or other skin-thinning medications or topical. • Injections within the last 7 days. <p><i>**Always follow your company guidelines on who should and should not receive a treatment**</i></p>

SERVICE TIPS
<ul style="list-style-type: none"> • It is strongly recommended to wear gloves during the equipment use portion of this protocol. • Adjust your suction and flow settings according to your guests needs.

POST TREATMENT INSTRUCTIONS
<ul style="list-style-type: none"> • Wear Sunscreen Daily • Resume your normal skin care next day • Avoid exfoliation for the next 7 days.

HEMOCARE RECOMMENDATIONS
<p>It is recommended guests go home with or replenish the following to maintain results</p> <ul style="list-style-type: none"> • Cleanser • Purifying and/or exfoliating product • Moisturizer

Detailed Protocol AquaBoost Signature Facial

STEPS	PRODUCT NAME	INSTRUCTIONS
CLEANSE	Prosana Refining Cleanser	<ul style="list-style-type: none"> Cleanse the skin using 1-2 pumps of the Prosana Refining Cleanser Once the skin has been cleansed thoroughly, remove the cleanser with a hot towel
TONE	Prosana Refining Mist OR Hydrating Mist	<p>Refining Mist:</p> <ul style="list-style-type: none"> For skin in need of added exfoliation or dull, lackluster skin. Apply with a cotton round or esthetic wipe avoiding the eye area. <p>Hydration Mist</p> <ul style="list-style-type: none"> For thirsty skin in need of added hydration. With eyes covered, spray directly over the face or use a cotton round or esthetic wipe to apply.
SERUM 1	Prosana Cleanse + Exfoliate	<ul style="list-style-type: none"> Turn on your AquaBoost equipment. Using the AquaBoost Serum 1, cleanse and exfoliate the skin. <p><i>*Please refer to the Owners Manual for usage instructions.</i></p>
SERUM 2	Prosana Deep Cleanse	<ul style="list-style-type: none"> Serum 2 is optional Perform another pass on the skin with Serum 2 to address excess sebum, or congestion. Commonly used in the T-Zone, around the mouth, chin and jaw, and around the nose. <p><i>*Please refer to the Owners Manual for usage instructions.</i></p>
SERUM 3	Prosana Soothe & Hydrate	<p>Turning the suction back down, and the serum flow up, hydrate the skin using Serum 3</p> <p><i>*Please refer to the Owners Manual for usage instructions.</i></p>

Detailed Protocol AquaBoost Signature Facial

STEPS	PRODUCT NAME	INSTRUCTIONS
MASK	Select Prosana Sheet Mask of your Choice	<p>Apply a Prosana mask that best suits your guests needs and leave on for 8 minutes.</p> <p>Recommended masks include:</p> <ul style="list-style-type: none"> • Hyaluronic Acid Sheet Mask • 4-in-1 Full Face Crystal Collagen Gel Mask • Vitamin C Crystal Collagen Gel Mask
HAND MASSAGE	Prosana Nail & Cuticle Oil	While your client is masking, perform a hand and nail massage.
REMOVE MASK		Gently remove the sheet mask.
CRYO WAND	Apply additional serum from within the Prosana Mask pouch .	<ul style="list-style-type: none"> • Feel the skin to determine if enough product remains on the skin to provide a slip for the Cryo Wand. If the skin is tacky, apply remaining serum from the mask package to the skin with hands for facial brush. • Using upward, circular motions, glide the Cryo-Wand over the skin moving from inside to the outside of the face. Spend extra time on any area of the skin that required additional extractions. • Do not make direct contact with the eyes.
MOISTURIZE	Prosana Calming Balm	If skin is in need of additional moisture, apply Calming Balm to the face, neck, and décolleté.



Treatment Preparation AquaBoost Facial

PRODUCTS REQUIRED FOR SERVICE:

SKU	PRODUCT NAME & PRODUCT AMOUNT
PSN130 OR PSN131	Serum 1: Cleanse & Exfoliate Serum 2: Deep Cleanse
PSN132	Serum 3: Soothe & Hydrate
AQB206	Treatment Tip
NB525	Gloves

CONTRAINDICATIONS

- Open lesions on the skin.
- Active Rosacea flare-up.
- Pregnant & nursing women should consult with a physician.
- Accutane or other skin-thinning medications or topical.
- Injections within the last 7 days.

****Always follow your company guidelines on who should and should not receive a treatment****

SERVICE TIPS

- The AquaBoost Facial Enhancement is intended to be added to an existing facial for added exfoliation, decongestion, and hydration.
- It is recommended that the AquaBoost Enhancement replace the exfoliation in your current protocol.
- The enhancement begins at the exfoliation step of your current protocol.
- It is strongly recommended to wear gloves during the service.

HEMOCARE RECOMMENDATIONS

It is recommended guests go home with or replenish the following to maintain results

- Cleanser
- Purifying and/or exfoliating product
- Moisturizer

POST TREATMENT INSTRUCTIONS

- Wear Sunscreen Daily
- Resume your normal skin care next day
- Avoid exfoliation for the next 7 days.

Detailed Protocol AquaBoost Facial

STEPS	PRODUCT NAME	INSTRUCTIONS
SKIN ANALYSIS	Select Cleanser of Your Choice	Analyze the skin and select serum <ul style="list-style-type: none"> • 1 - Dry, Normal, Combination Skin Serum • 2 - Oily, Congested, Acneic / Acne Prone Skin
SERUM 1 or SERUM 2	Cleanse & Exfoliate or Deep Cleanse	Perform 1 pass with Serum 1 or 2: <ul style="list-style-type: none"> • Beginning at the chin, move in upward and outward motions. • Hold the skin taut with one hand while gliding with the other to ensure a smooth glide without skipping or rapid release. • Follow the grid provided in the Training Manual for added guidance. • Spend additional time as needed in areas of increased congestion. *Approximately 3 fl. oz of serum will be used.
SERUM 3	Soothe & Hydrate	Perform 1 pass with Serum 3: <ul style="list-style-type: none"> • Press excess serum into the skin throughout the service. *Approximately 2-3 fl. oz will be used.
CRYOWAND	Select Gel Mask or Heavy Gel Serum	Perform 1 pass with CryoWand <ul style="list-style-type: none"> • Begin at the chin, move in circular motions upward and outward. • Keep the CryoWand moving at all times.

Treatment Preparation AquaBoost Exfoliation Boost

PRODUCTS REQUIRED FOR SERVICE:

SKU	PRODUCT NAME & PRODUCT AMOUNT
PSN130	Serum 1: Cleanse & Exfoliate
PSN131	Serum 2: Deep Cleanse
AQB206	Treatment Tip
NB525	Gloves

CONTRAINDICATIONS

- Open lesions on the skin.
- Active Rosacea flare-up.
- Pregnant & nursing women should consult with a physician.
- Accutane or other skin-thinning medications or topical gels/creams.
- Injections within the last 7 days.

****Always follow your company guidelines on who should and should not receive a treatment****

SERVICE TIPS

- The AquaBoost Exfoliation Enhancement is intended to be added to an existing facial for added exfoliation, decongestion, and hydration.
- It is recommended that this AquaBoost Enhancement replaces the exfoliation in your current protocol.
- The enhancement begins at the exfoliation step of your current protocol.
- Prepares the skin to receive intense hydration by exfoliating the skin and removing impurities safely and gently.
- It is strongly recommended to wear gloves during the service.

HEMOCARE RECOMMENDATIONS

It is recommended guests go home with or replenish the following to maintain results

- Cleanser
- Purification and/or Exfoliation Product
- Targeted Treatment
- Moisturizer

POST TREATMENT INSTRUCTIONS

- Wear Sunscreen Daily
- Resume your normal skin care next day
- Avoid exfoliation for the next 7 days.



Detailed Protocol AquaBoost Exfoliation Boost

STEP	PRODUCT NAME	INSTRUCTIONS
SKIN ANALYSIS		<p>Analyze the skin and select serum</p> <ul style="list-style-type: none"> • Serum 1 – Dry, Normal, Combination Skin • Serum 2 – Oily, Congested, Acneic / Acne Prone Skin
SERUM 1 OR SERUM 2	Cleanse & Exfoliate OR Deep Cleanse	<p>Perform 1 pass with Serum 1 or 2:</p> <ul style="list-style-type: none"> • Beginning at the chin, move in upward and outward motions. • Hold the skin taut with one hand while gliding with the other to ensure a smooth glide without skipping or rapid release. • Follow the grid provided in the Training Manual for added guidance. • Spend additional time as needed in areas of increased congestion. <p>*Approximately 3 fl. oz of serum will be used.</p>
CRYOWAND	Select Gel Mask or Heavy Gel Serum	<p>Perform 1 pass with the CryoWand</p> <ul style="list-style-type: none"> • Begin at the chin, move in circular motions upward and outward. • Keep the CryoWand moving at all times.



Treatment Preparation AquaBoost Back Facial

PRODUCTS REQUIRED FOR SERVICE:

SKU	PRODUCT NAME & PRODUCT AMOUNT
PSN130 OR PSN131	Serum 1: Cleanse & Exfoliate Serum 2: Deep Cleanse
PSN132	Serum 3: Soothe & Hydrate
AQB206	Treatment Tip
NB525	Gloves

CONTRAINDICATIONS
<ul style="list-style-type: none"> • Open lesions on the skin. • Pregnant & nursing women should consult with a physician. • Accutane or other skin-thinning medications or topical gels/creams. <p><i>**Always follow your company guidelines on who should and should not receive a treatment**</i></p>

SERVICE TIPS
<ul style="list-style-type: none"> • Guests may find their sides to be sensitive and prefer not to have their sides included in the treatment. • The AquaBoost Back Facial is suitable for the entire back however the 60-minute protocol includes the Mid Back, Upper Back, & Shoulders only in regards to product quantity. • It is appropriate to perform the back facial by spot treating versus the entire back if necessary.

HOMECARE RECOMMENDATIONS
<p>It is recommended guests go home with or replenish the following to maintain results</p> <ul style="list-style-type: none"> • Purifying and/or Exfoliating Cleanser • Spot treatment for acne challenges

POST TREATMENT INSTRUCTIONS
<ul style="list-style-type: none"> • Wear Sunscreen Daily • Resume your normal skin care next day • Avoid exfoliation for the next 7 days. • Schedule next visit.

Detailed Protocol AquaBoost Back Facial

STEPS	PRODUCT NAME	INSTRUCTIONS
Cleanse	Select Cleanser of Your Choice	Thoroughly cleanse the back to prepare the skin for the AquaBoost treatment.
Exfoliate	Serum 1 Cleanse & Exfoliate	<ul style="list-style-type: none"> • Work from the inside to the outside of the back. (ie from the spine to the side) • Hold the skin taut with one hand while gliding the treatment wand with the other. This will help avoid skipping and rapid release of the wand. • Work in sections from inside to outside completing: <ul style="list-style-type: none"> Mid Back Upper Back & Shoulders <p>**The full back can receive the AquaBoost service, however will require additional serum than accounted for in the protocol. It is also feasible to spot-treat specific areas such as upper back only, only acne challenged locations, etc.**</p>
Extractions	Serum 2 Deep Cleanse	<ul style="list-style-type: none"> • If your guest requires extractions or excessive sebum removal, setup Serum 2. • Perform passes from the inside to the outside of the back in the areas needing extractions and deep cleansing. • Use the twist method on stubborn blemishes to help with extraction. <p>**Apply Desincrustation Solution to areas with more congestion or stubborn extraction sites to help prep the skin. Apply solution prior to setting up Serum 2 to allow time to take action.**</p> <p>**Perform manual extractions if necessary at this time.**</p>
Hydrate	Serum 3 Soothe & Hydrate	<ul style="list-style-type: none"> • Work from the inside to the outside of the back. (ie from the spine to the side) • Hold the skin taut with one hand while gliding the treatment wand with the other. This will help avoid skipping and rapid release of the wand. • Work in sections from inside to outside completing: <ol style="list-style-type: none"> 1. Mid Back 2. Upper Back & Shoulders • Press any serum into the skin that is not immediately absorbed to avoid dripping.
CryoWand	Select Gel Mask or Heavy Gel Serum	<p>Perform 1 pass with CryoWand</p> <ul style="list-style-type: none"> • Turn on the CryoWand • Alert the guest they will experience a cold feeling from the CryoWand • Apply Gel Mask / Heavy Gel Serum • Work in sections from inside to outside completing: <ol style="list-style-type: none"> a. Mid Back b. Upper Back & Shoulders • Move in circular motions upward and outward. • Keep the CryoWand moving at all times. • Remove excess product with a warm towel.
Moisturize	Select Moisturizer of your choice	<ul style="list-style-type: none"> • Apply additional moisturizer if needed.



Treatment Preparation AquaBoost Fanny Facial

PRODUCTS REQUIRED FOR SERVICE:

SKU	PRODUCT NAME & PRODUCT AMOUNT
PSN130 OR PSN131	Serum 1: Cleanse & Exfoliate Serum 2: Deep Cleanse
PSN132	Serum 3: Soothe & Hydrate
AQB206	Treatment Tip
NB525	Gloves
PSN131 (Optional)	Serum 2: Deep Cleanse

CONTRAINDICATIONS
<ul style="list-style-type: none"> • Open lesions on the skin. • Active Rosacea flare-up. • Pregnant & nursing women should consult with a physician. • Accutane or other skin-thinning medications or topical. <p>**Always follow your company guidelines on who should and should not receive a treatment**</p>

SERVICE TIPS
<ul style="list-style-type: none"> • Please follow local and state guidelines on draping and treatment locations policies. • It is strongly recommended that you wear gloves throughout the entire treatment.

HOMECARE RECOMMENDATIONS
<p>It is recommended guests go home with or replenish the following to maintain results</p> <ul style="list-style-type: none"> • Cleanser • Purifying and/or exfoliating product • Moisturizer

POST TREATMENT INSTRUCTIONS
<ul style="list-style-type: none"> • Wear Sunscreen Daily • Resume your normal skin care next day • Avoid exfoliation for the next 7 days.

Detailed Protocol AquaBoost Fanny Facial

STEPS	PRODUCT NAME	INSTRUCTIONS
CLEANSE	Select Cleanser of Your Choice	<ul style="list-style-type: none"> Thoroughly cleanse the fanny to prepare the skin for the AquaBoost treatment. Steam is optional during cleansing. <p>**Disposable panties are recommended for modest guests</p> <p>**For states that allow fully uncovered fannies, a rolled towel vertically along the center of the glutes is recommended, allowing both glutes to be fully exposed for treatment.**</p>
SERUM 1	Cleanse & Exfoliate	<ul style="list-style-type: none"> Hold the skin taut with one hand while gliding the treatment wand with the other. This will help avoid skipping and rapid release of the wand. Perform one pass: <ul style="list-style-type: none"> Begin on the left. Pull the treatment wand vertically from the bottom to the top of the glute. Repeat one pass on the right glute. <p>For areas of congestion, repeat a pass horizontally in that section.</p>
SERUM 2 OPTIONAL	Deep Cleanse	<ul style="list-style-type: none"> Spot treat acne and areas of congestion using Serum 2 Use small vertical and horizontal strokes Use the twist method on stubborn blemishes to help with extraction. <p>**Apply Desincrustation Solution to areas with more congestion or stubborn extraction sites to help prep the skin. Apply solution prior to setting up Serum 2 to allow time to take action.**</p>
SERUM 3	Soothe & Hydrate	<ul style="list-style-type: none"> Hold the skin taut with one hand while gliding the treatment wand with the other. This will help avoid skipping and rapid release of the wand. Perform one pass: <ul style="list-style-type: none"> Begin on the left. Pull the treatment wand vertically from the bottom to the top of the glute. Repeat one pass on the right glute. Press any serum into the skin that is not immediately absorbed to avoid dripping.
CRYOWAND	Select Gel Mask or Heavy Gel Serum	<p>Perform 1 pass with CryoWand</p> <ul style="list-style-type: none"> Turn on the CryoWand. Alert the guest they will experience a cold feeling from the CryoWand. Apply Gel Mask / Heavy Gel Serum. Begin on the left glute rotating in circular motions, up the glute. Keep the CryoWand moving at all times. Remove excess product with a warm towel.
MOISTURIZE	Select Moisturizer of your choice	<ul style="list-style-type: none"> Apply additional moisturizer if needed.



Treatment Preparation AquaBoost Hand Rejuvenation

PRODUCTS REQUIRED FOR SERVICE:

SKU	PRODUCT NAME & PRODUCT AMOUNT
PSN130	Serum 1: Cleanse & Exfoliate
PSN132	Serum 3: Soothe & Hydrate
AQB206	Treatment Tip
NB525	Gloves

CONTRAINDICATIONS
<ul style="list-style-type: none"> • Open lesions on the skin. • Active Rosacea flare-up. • Pregnant & nursing women should consult with a physician. • Accutane or other skin-thinning medications or topical. <p>**Always follow your company guidelines on who should and should not receive a treatment**</p>

SERVICE TIPS
<ul style="list-style-type: none"> • It is strongly recommended to wear gloves during the service.

HEMOCARE RECOMMENDATIONS
<p>It is recommended guests go home with or replenish the following to maintain results</p> <ul style="list-style-type: none"> • Hand Cream

POST TREATMENT INSTRUCTIONS
<ul style="list-style-type: none"> • Wear Sunscreen Daily • Resume your normal skin care next day • Avoid exfoliation for the next 7 days.



Detailed Protocol AquaBoost Hand Rejuvenation

STEPS	PRODUCT NAME	INSTRUCTIONS
CLEANSE - LEFT HAND		<ul style="list-style-type: none"> Cleanse the left hand with a warm towel.
SERUM 1	Cleanse & Exfoliate	<ul style="list-style-type: none"> Perform 1 pass with Serum 1 Hold the skin taut with one hand while gliding with the other to help ensure a smooth glide without skipping or rapid release. <p>Top of Hand:</p> <ul style="list-style-type: none"> Glide from the wrist down to the Metacarpal Bone Joints, working from the inside of the hand to the outside of the hand. Take caution over the Metacarpal Bones to not add too much pressure with the treatment wand. Glide down the top of each finger. Spend additional time as needed in areas of greater keratinization. <p>Palm of Hand (optional)</p> <ul style="list-style-type: none"> Repeat top of hand pattern. Can be performed on the entire palm of hand or used to treat calluses only. <p><i>*Approximately 2 fl. oz of serum will be used per hand.*</i></p>
SERUM 3	Soothe & Hydrate	<p>Perform 1 pass with Serum 3:</p> <ul style="list-style-type: none"> Follow the same glide patterns as you did for Serum 1. Press any excess serum into the hand as you proceed. <p><i>*Approximately 2 fl. oz will be used per hand.*</i></p>
CRYOWAND	Gel Mask or Serum of your choice	<ul style="list-style-type: none"> Turn on the CryoWand. Apply a thin layer of mask to the top of the hand. Alert the guest they will experience a cold feeling from the CryoWand. Use circular motions across the hand. Remove excess product with a dry towel.
MOISTURIZE	Moisturizer of your choice	<ul style="list-style-type: none"> Using a product of your choice, apply moisturizer to the hand and wrist. Perform brief hand massage. Place hand under draping when finished.
REPEAT ON RIGHT HAND		<ul style="list-style-type: none"> Repeat all steps on the right hand.



AquaBoost Post Service Cleaning Protocol

Clean after every service to avoid clogging and deposit of any kind.
 Cleaning your AquaBoost properly is essential for it's proper function and warranty coverage.

STEPS	INSTRUCTIONS
1	Replace clear treatment tip on handpiece with blue cleaning tip.
2	To perform cleaning, remove Serum 1 bottle. Fill the white/opaque cleaning solution bottle with room temperature or warm distilled water.
3	Install the hose and cap from the Serum 1 bottle on the cleaning bottle and connect to the S1 port on the top of the machine.
4	Open the flow control knob on the side of the machine to the fully open position.
5	Turn on the S1 serum switch (press twice until it lights up) and run a full bottle of warm distilled water through the S1 side.
6	Repeat steps on the S3 side.
7	When complete, disconnect the cleaning bottle and leave disconnected to air dry in a sterile location.
8	Cleaning steps should be completed after each use.
PLEASE NOTE YOUR WARRANTY WILL BE VOID IF CLEANING IS NOT PERFORMED AFTER EACH SESSION.	

PRO TIPS:

**Run the distilled water while you are cleaning your bowls and brushes to save time.
 Have 2 bottles of cleaning water prepared prior to your service to save time.**



AquaBoost End of Day Cleaning Protocol

Clean after every service to avoid clogging and deposit of any kind.
Cleaning your AquaBoost properly is essential for it's proper function and warranty coverage.

STEPS	INSTRUCTIONS
1	Replace clear treatment tip on handpiece with blue cleaning tip.
2	To perform cleaning, remove Serum 1 bottle. Fill the white/opaque cleaning solution bottle with warm distilled water and vinegar. (up to 10% distilled vinegar only)
3	Install the hose and cap from the Serum 1 bottle on the cleaning bottle and connect to the S1 port on the top of the machine.
4	Open the flow control knob on the side of the machine to the fully open position.
5	Turn on the S1 serum switch (press twice until it lights up) and run a full bottle of warm distilled water through the S1 side.
6	Repeat steps on the S3 side.
7	After running a full bottle of warm distilled water and vinegar solution through S1 and S3 sides, disconnect the cleaning bottle and leave disconnected to air dry in a sterile area.
PLEASE NOTE YOUR WARRANTY WILL BE VOID IF PROPER CLEANING IS NOT MAINTAINED	

PRO TIPS:

Run the cleaning solution while you are cleaning your bowls and brushes to save time.
Have 2 bottles of cleaning solution prepared prior to your service to save time.

SAFETY DATA SHEET

Section 1. Identification

Product Name: Prosana Aquaboost Serum 1: Cleanse & Exfoliate

Product Number: PSN130

Distributor Name: Universal Companies, Inc.
18260 Oak Park Drive
Abingdon, VA 24210

Emergency Contact: (888) 558 5571

Recommended Use: For use with AquaBoost hydrodermabrasion equipment (AQB200)

Restrictions On Use: None.

Section 2. Hazards Identification

Emergency Overview

This product is: Clear Liquid

Water Solubility: Soluble

Odor is: Slight Floral

Flash Point: N/A

Flammability: None

OSHA Physical Hazard: None

Potential Acute Health Hazards

None

Potential Chronic Health Hazards

None

Section 3. Composition / Information on Ingredients

No ingredients listed on OSHA Occupational Chemical Database

SAFETY DATA SHEET

Section 4. First Aid Measures

Emergency and First Aid Measures

- Eyes: Irrigate thoroughly with plenty of water until discomfort is relieved.
- Skin: No evidence of adverse effects. If irritation occurs wash thoroughly with cold water and mild soap.
- Inhalation: No evidence of adverse effects
- Ingestion: If a large amount is swallowed, get medical attention
-

Section 5. Fire Fighting Measures

- Flashpoint: N/A
- Flammability: None
- Extinguishing Media: Dry Chemical, Foam or Carbon Dioxide
- Special Fire Fighting Procedure: None
- Unusual Fire and Explosion Hazards: None
-

Section 6. Accidental Release Measures

Cover with an inert absorbent material and remove to a disposal container. Mop up and then wash with soapy water.

- Disposal Procedure: Dispose of as specified by local codes. Observe local, state and federal regulations.
-

Section 7. Handling & Storage

Covered storage at 15-25°C

Avoid freezing temperatures

- Flammability: None

SDS – Prosana Aquaboost Serum 1: Cleanse & Exfoliate - 2

SAFETY DATA SHEET

Other Physical Hazards: None

Section 8. Exposure Controls / Personal Protection

Ingredients with limit values that require monitoring at the workplace: None

Personal Protective Measures:

Respiratory Protection: None

Ventilation Protection: None

Eye protection: None

Protective Gloves: None

Section 9. Physical and Chemical Properties

PARAMETER	VALUE
Appearance:	Clear Liquid
Colour:	None
Odor:	Slight Floral
Physical State:	Liquid
pH:	4.0 – 4.5
Boiling Point:	100C
Vapor Density	Not applicable
Solubility in Water:	Soluble
Viscosity:	~1cP
% VOC	Complies with Federal and State regulations

SAFETY DATA SHEET

Section 10. Stability and Reactivity

Stability:	Stable
Incompatibility:	None
Hazardous Decomposition Products	None
Hazardous polymerization:	Will Not Occur

Section 11. Toxicological Information

Eye Irritation:	Mild eye irritant
Skin Corrosion/Irritation:	Not a skin irritant based on product testing
Respiratory or Skin Sensitization:	Not a skin sensitizer based on product testing
Acute toxicity:	Not Available
Germ Cell Mutagenicity:	Does not contain any ingredients known to be mutagenic according to OSHA
Carcinogenicity:	Does not contain any ingredients known to be carcinogenic according to OSHA
Reproductive toxicity:	Not expected to cause reproductive toxicity
STOT – Single Exposure:	Not expected to cause organ toxicity
STOT – Repeat Exposure:	Not expected to cause organ toxicity
Aspiration Hazard:	Not an aspiration hazard

Section 12. Ecological Information

Aquatic Toxicity:	Not expected to cause aquatic toxicity
Persistence and Degradability:	Not Available
Bioaccumulative Potential:	Not Available
Mobility in Soil:	Not Available

SDS – Prosana Aquaboost Serum 1: Cleanse & Exfoliate - 4

SAFETY DATA SHEET

Other Adverse Effects: Not expected to cause adverse effects.

Section 13. Disposal Considerations

Consult applicable Local and State Regulations for disposal of non-hazardous liquids or solids. Do not dispose down sewer systems. Disposal in a properly regulated landfill is acceptable.

Section 14. Transport Information

Ground (DOT): Not hazardous for transport.

Vessel (IMDG): Not hazardous for transport.

Air (IATA): Not hazardous for transport.

Section 15. Regulatory Information

US Federal

The product described in this Safety Data Sheet is regulated under the Federal Food, Drug, and Cosmetics Act and is safe to use as per directions on container, box or accompanying literature (where applicable).

US States

CA Prop 65: This product is not subject to warning labeling under California Proposition 65.

State Right-to-Know: No ingredients present in the finished product are listed on state right-to-know lists or state work exposure lists.

Toxic Substances Control Act (TSCA) Inventory Status:

This material is for cosmetic use.

TSCA Export Notification Rules:

This product does not contain any TSCA export notifiable ingredients.

Sara Title III Section 313

This product contains no chemicals subject to the reporting requirement of section 313 of SARA.

CERCLA 302.4 Hazardous Substances

Contains no CERCLA reportable ingredients.

SDS – Prosana Aquaboost Serum 1: Cleanse & Exfoliate - 5

SAFETY DATA SHEET

Section 16. Other Information

These safety guidelines are based on information obtained from reliable sources. Prudent use and handling as part of good manufacturing practices is advised.

Date of Preparation: 31st August 2022

SAFETY DATA SHEET

Section 1. Identification

Product Name: Prosana Aquaboost Serum 2: Deep Cleanse

Product Number: PSN130

Distributor Name: Universal Companies, Inc.
18260 Oak Park Drive
Abingdon, VA 24210

Emergency Contact: (888) 558 5571

Recommended Use: For use with AquaBoost hydrodermabrasion equipment (AQB200)

Restrictions On Use: None.

Section 2. Hazards Identification

Emergency Overview

This product is: Clear Liquid

Water Solubility: Soluble

Odor is: Characteristic

Flash Point: N/A

Flammability: None

OSHA Physical Hazard: None

Potential Acute Health Hazards

None

Potential Chronic Health Hazards

None

Section 3. Composition / Information on Ingredients

No ingredients listed on OSHA Occupational Chemical Database

SAFETY DATA SHEET

Section 4. First Aid Measures

Emergency and First Aid Measures

- Eyes: Irrigate thoroughly with plenty of water until discomfort is relieved.
- Skin: No evidence of adverse effects. If irritation occurs wash thoroughly with cold water and mild soap.
- Inhalation: No evidence of adverse effects
- Ingestion: If a large amount is swallowed, get medical attention
-

Section 5. Fire Fighting Measures

- Flashpoint: N/A
- Flammability: None
- Extinguishing Media: Dry Chemical, Foam or Carbon Dioxide
- Special Fire Fighting Procedure: None
- Unusual Fire and Explosion Hazards: None
-

Section 6. Accidental Release Measures

Cover with an inert absorbent material and remove to a disposal container. Mop up and then wash with soapy water.

- Disposal Procedure: Dispose of as specified by local codes. Observe local, state and federal regulations.
-

Section 7. Handling & Storage

Covered storage at 15-25°C

Avoid freezing temperatures

- Flammability: None

SDS – Prosana Aquaboost Serum 2: Deep Cleanse - 2

SAFETY DATA SHEET

Other Physical Hazards: None

Section 8. Exposure Controls / Personal Protection

Ingredients with limit values that require monitoring at the workplace: None

Personal Protective Measures:

Respiratory Protection: None

Ventilation Protection: None

Eye protection: None

Protective Gloves: None

Section 9. Physical and Chemical Properties

PARAMETER	VALUE
Appearance:	Clear Liquid
Colour:	None
Odor:	Characteristic
Physical State:	Liquid
pH:	4.0 – 5.0
Boiling Point:	100C
Vapor Density	Not applicable
Solubility in Water:	Soluble
Viscosity:	~1cP
% VOC	Complies with Federal and State regulations

SAFETY DATA SHEET

Section 10. Stability and Reactivity

Stability:	Stable
Incompatibility:	None
Hazardous Decomposition Products	None
Hazardous polymerization:	Will Not Occur

Section 11. Toxicological Information

Eye Irritation:	Mild eye irritant
Skin Corrosion/Irritation:	Not a skin irritant based on product testing
Respiratory or Skin Sensitization:	Not a skin sensitizer based on product testing
Acute toxicity:	Not Available
Germ Cell Mutagenicity:	Does not contain any ingredients known to be mutagenic according to OSHA
Carcinogenicity:	Does not contain any ingredients known to be carcinogenic according to OSHA
Reproductive toxicity:	Not expected to cause reproductive toxicity
STOT – Single Exposure:	Not expected to cause organ toxicity
STOT – Repeat Exposure:	Not expected to cause organ toxicity
Aspiration Hazard:	Not an aspiration hazard

Section 12. Ecological Information

Aquatic Toxicity:	Not expected to cause aquatic toxicity
Persistence and Degradability:	Not Available
Bioaccumulative Potential:	Not Available
Mobility in Soil:	Not Available

SDS – Prosana Aquaboost Serum 2: Deep Cleanse - 4

SAFETY DATA SHEET

Other Adverse Effects: Not expected to cause adverse effects.

Section 13. Disposal Considerations

Consult applicable Local and State Regulations for disposal of non-hazardous liquids or solids. Do not dispose down sewer systems. Disposal in a properly regulated landfill is acceptable.

Section 14. Transport Information

Ground (DOT): Not hazardous for transport.

Vessel (IMDG): Not hazardous for transport.

Air (IATA): Not hazardous for transport.

Section 15. Regulatory Information

US Federal

The product described in this Safety Data Sheet is regulated under the Federal Food, Drug, and Cosmetics Act and is safe to use as per directions on container, box or accompanying literature (where applicable).

US States

CA Prop 65: This product is not subject to warning labeling under California Proposition 65.

State Right-to-Know: No ingredients present in the finished product are listed on state right-to-know lists or state work exposure lists.

Toxic Substances Control Act (TSCA) Inventory Status:

This material is for cosmetic use.

TSCA Export Notification Rules:

This product does not contain any TSCA export notifiable ingredients.

Sara Title III Section 313

This product contains no chemicals subject to the reporting requirement of section 313 of SARA.

CERCLA 302.4 Hazardous Substances

Contains no CERCLA reportable ingredients.

SDS – Prosana Aquaboost Serum 2: Deep Cleanse - 5

SAFETY DATA SHEET

Section 16. Other Information

These safety guidelines are based on information obtained from reliable sources. Prudent use and handling as part of good manufacturing practices is advised.

Date of Preparation: 31st August 2022



SAFETY DATA SHEET

Section 1. Identification

Product Name: Prosana Aquaboost Serum 3: Soothe & Hydrate
Product Number: PSN130
Distributor Name: Universal Companies, Inc.
18260 Oak Park Drive
Abingdon, VA 24210
Emergency Contact: (888) 558 5571
Recommended Use: For use with AquaBoost hydrodermabrasion equipment
(AQB200)
Restrictions On Use: None.

Section 2. Hazards Identification

Emergency Overview

This product is: Clear Liquid
Water Solubility: Soluble
Odor is: Slight Floral
Flash Point: N/A
Flammability: None
OSHA Physical Hazard: None

Potential Acute Health Hazards

None

Potential Chronic Health Hazards

None

Section 3. Composition / Information on Ingredients

No ingredients listed on OSHA Occupational Chemical Database

SAFETY DATA SHEET

Section 4. First Aid Measures

Emergency and First Aid Measures

- Eyes: Irrigate thoroughly with plenty of water until discomfort is relieved.
- Skin: No evidence of adverse effects. If irritation occurs wash thoroughly with cold water and mild soap.
- Inhalation: No evidence of adverse effects
- Ingestion: If a large amount is swallowed, get medical attention
-

Section 5. Fire Fighting Measures

- Flashpoint: N/A
- Flammability: None
- Extinguishing Media: Dry Chemical, Foam or Carbon Dioxide
- Special Fire Fighting Procedure: None
- Unusual Fire and Explosion Hazards: None
-

Section 6. Accidental Release Measures

Cover with an inert absorbent material and remove to a disposal container. Mop up and then wash with soapy water.

- Disposal Procedure: Dispose of as specified by local codes. Observe local, state and federal regulations.
-

Section 7. Handling & Storage

Covered storage at 15-25°C

Avoid freezing temperatures

- Flammability: None

SDS – Prosana Aquaboost Serum 3: Soothe & Hydrate - 2

SAFETY DATA SHEET

Other Physical Hazards: None

Section 8. Exposure Controls / Personal Protection

Ingredients with limit values that require monitoring at the workplace: None

Personal Protective Measures:

Respiratory Protection: None

Ventilation Protection: None

Eye protection: None

Protective Gloves: None

Section 9. Physical and Chemical Properties

PARAMETER	VALUE
Appearance:	Clear Liquid
Colour:	None
Odor:	Slight Floral
Physical State:	Liquid
pH:	4.5 – 5.0
Boiling Point:	100C
Vapor Density	Not applicable
Solubility in Water:	Soluble
Viscosity:	~1cP
% VOC	Complies with Federal and State regulations

SAFETY DATA SHEET

Section 10. Stability and Reactivity

Stability:	Stable
Incompatibility:	None
Hazardous Decomposition Products	None
Hazardous polymerization:	Will Not Occur

Section 11. Toxicological Information

Eye Irritation:	Mild eye irritant
Skin Corrosion/Irritation:	Not a skin irritant based on product testing
Respiratory or Skin Sensitization:	Not a skin sensitizer based on product testing
Acute toxicity:	Not Available
Germ Cell Mutagenicity:	Does not contain any ingredients known to be mutagenic according to OSHA
Carcinogenicity:	Does not contain any ingredients known to be carcinogenic according to OSHA
Reproductive toxicity:	Not expected to cause reproductive toxicity
STOT – Single Exposure:	Not expected to cause organ toxicity
STOT – Repeat Exposure:	Not expected to cause organ toxicity
Aspiration Hazard:	Not an aspiration hazard

Section 12. Ecological Information

Aquatic Toxicity:	Not expected to cause aquatic toxicity
Persistence and Degradability:	Not Available
Bioaccumulative Potential:	Not Available
Mobility in Soil:	Not Available

SDS – Prosana Aquaboost Serum 3: Soothe & Hydrate - 4

SAFETY DATA SHEET

Other Adverse Effects: Not expected to cause adverse effects.

Section 13. Disposal Considerations

Consult applicable Local and State Regulations for disposal of non-hazardous liquids or solids. Do not dispose down sewer systems. Disposal in a properly regulated landfill is acceptable.

Section 14. Transport Information

Ground (DOT): Not hazardous for transport.

Vessel (IMDG): Not hazardous for transport.

Air (IATA): Not hazardous for transport.

Section 15. Regulatory Information

US Federal

The product described in this Safety Data Sheet is regulated under the Federal Food, Drug, and Cosmetics Act and is safe to use as per directions on container, box or accompanying literature (where applicable).

US States

CA Prop 65: This product is not subject to warning labeling under California Proposition 65.

State Right-to-Know: No ingredients present in the finished product are listed on state right-to-know lists or state work exposure lists.

Toxic Substances Control Act (TSCA) Inventory Status:

This material is for cosmetic use.

TSCA Export Notification Rules:

This product does not contain any TSCA export notifiable ingredients.

Sara Title III Section 313

This product contains no chemicals subject to the reporting requirement of section 313 of SARA.

CERCLA 302.4 Hazardous Substances

Contains no CERCLA reportable ingredients.

SDS – Prosana Aquaboost Serum 3: Soothe & Hydrate - 5

SAFETY DATA SHEET

Section 16. Other Information

These safety guidelines are based on information obtained from reliable sources. Prudent use and handling as part of good manufacturing practices is advised.

Date of Preparation: 31st August 2022



The UCo Difference



We are a full solutions provider.



We read labels.



We train and educate.



We are staffed by pros.

Universal  Companies.

universalcompanies.com | 1.800.558.5571

