

Cleaning Procedure Check List

- 1. Remove clear serum tip on handheld wand
- 2. Place the blue cleaning tip on handheld wand
- 3. Fill the cleaning solution bottle with warm distilled water and vinegar (up to 10% vinegar only)
- 4. Remove the cap with hose from the Serum 1 bottle and replace with its original lid. Do not remove the tube from the serum suction tube ports
- 5. Secure the Serum 1 cap with hose onto the cleaning solution bottle
- 6. Adjust the flow control knob on the side of the machine to the fully open position
- 7. Adjust the pressure control knob on the side of the machine to the fully open position
- 8. Turn on the S1 serum switch (press twice until it lights up) and allow full bottle of cleaning solution to flow through the S1 side
- 9. Remove the cap with hose from the cleaning solution bottle and remove the hose from the serum suction tube port. Set aside to air dry
- 10. If necessary to prevent overflowing, drain and rinse the waste collection container before repeating the steps for the S3 side and reattach
- 11. Repeat Steps 1-9 on the S3 side
- 12. Drain the waste collection container and wash thoroughly with antibacterial soap and allow to air dry if time allows between clients or do a final deep clean of the piece at the end of the day
- 13. Wipe down the machine with disinfecting wipes including the handheld wand and CryoWand. CryoWand can be cleaned with an alcohol wipe on the metal portion